


## Instructions for Installing Tubestand Interlock Bracket


(Store this sheet with the table manual for future reference.)

This tubestand/grid cabinet interlock bracket will bolt to the rear of the grid cabinet using the hardware provided. The spring-loaded tubestand interlock pivots to engage or disengage with the tubestand as shown in Figures 1 and 2. Adjustment nuts accessed from the underside of the interlock bracket control the spacing between the grid cabinet and tubestand to ensure a good fit. Insert the two-pin cable plug into the mated receptacle at the rear of the grid cabinet.


When the interlock bracket is swiveled forward to engage the tubestand a microswitch is activated, disabling the grid cabinet lock and allowing the grid cabinet to move in tandem with the tubestand. If the operator wishes the tubestand and grid cabinet to move independently, disengage the interlock bracket from the tubestand. This will re-activate the grid cabinet lock.

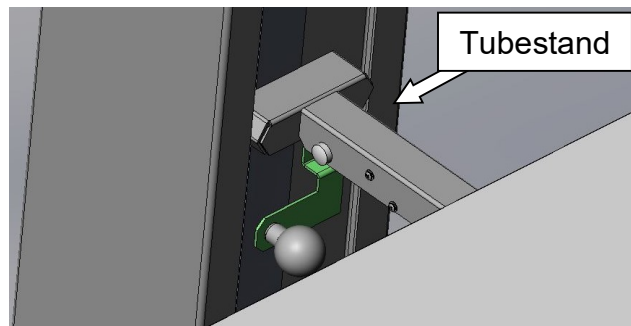

**Figure 1 - Interlock Engaged**

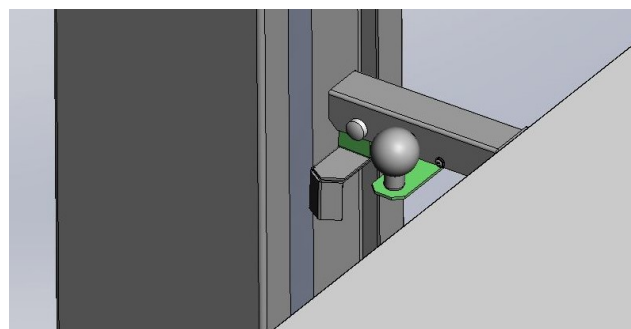

**Figure 2 - Interlock Disengaged**

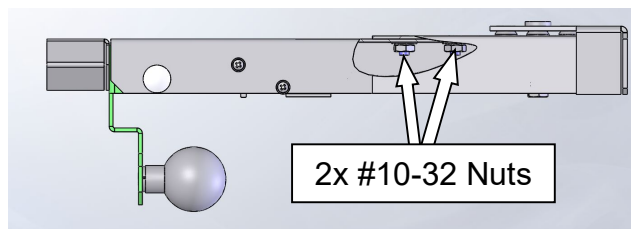

**Figure 3 - Adjustment Nuts**