# **Challenge-X**

**X-ray System**

**Service Manual**

*Technical Publication*  
*SM-1236R0\_EN\_SED*  
SUMMIT P/N 09530-002 REV. A

# **Service Manual**

## **Challenge-X** **X-ray System**

**This manual covers the following equipments / Este manual cubre los siguientes equipos**  
**Ce manuel couvre les équipements suivants / Il presente manuale descrive i seguenti dispositivi**

**X-ray System Challenge-X:  
CHALLENGEX AP Radiographic System, composed of:  
Ceiling Suspension CHALLENGEX: CHAP  
Elevating Table NET500: NET500AP  
Wall Stand CHALLENGEX: CHWSAP**

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## **IMPORTANT ! ... X-ray Protection**

### **X-RAY EQUIPMENT IS DANGEROUS TO BOTH PATIENT AND OPERATOR UNLESS MEASURES OF PROTECTION ARE STRICTLY OBSERVED**

**X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein should be thoroughly read and understood before attempting to place this equipment in operation. We will be glad to assist and cooperate in placing this equipment in use.**

Although this apparatus is built to the highest safety standards and incorporates a high degree of protection against X-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly, unwisely, or unknowingly exposing themselves or others to X-radiation.

It is important that everyone working with X-radiation be properly trained and take adequate steps to insure protection against injury.

The manufacturer assumes that all operator and service personnel authorized to use, install, calibrate and maintain this equipment is cognizant of the danger of excessive exposure to X-radiation, is sufficiently trained and has the required knowledge for it. The equipment herein described is sold with the understanding that the manufacturer, its agents, and representatives are not liable for injury or damage which may result from exposure to X-radiation.

Various protective material and devices are available. It is recommended that such materials and devices be used.

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---

## **LIST OF DOCUMENTS**

This Service Manual comprises the following documents:

---

<b>CODE / REVISION</b>	<b>DOCUMENT</b>
IN-1236R0	Installation
CC-1236R0	Calibration & Configuration
TR-1236R0	Troubleshooting
FR-1236R0	Field Replacement
MA-1236R0	Maintenance
SC-1236R0	Schematics
RP-1236R0	Renewal Parts

---