




VX_{VUE} Integration

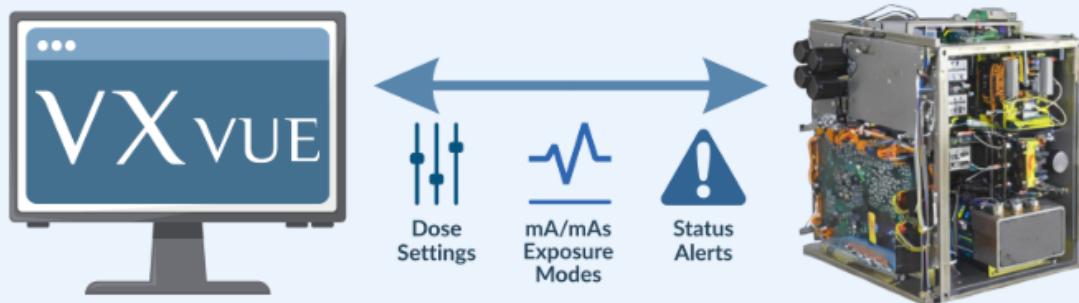

Hey there Subscriber,

VXvue supports integration with **Sedecal (SHFR) generators** using the GS.SEDECAL Generator Service, allowing direct communication between the system and generator via RS-232C.

Key capabilities include:

- Transfer of technique settings between VXvue & the generator
- Support for **AEC, mAs, and mA/ms exposure modes**
- Display of exposed dose values, generator status, & warnings
- Setting pre-defined APRs through VXvue's Procedure Manager

Integration requirements:

- VXvue version 1.0.0.87p4 or higher
- GS.SEDECAL Generator Service (v1.0.0.2)
- Proper port configuration and verification using the Echo function in VXSetup

This integration helps reduce manual setup, improves consistency during exams, and supports more efficient system operation.

See the document below for more detailed instruction. [!\[\]\(cf531ed27e91483460120fcc057b3901_img.jpg\)](#)

How to Integrate VXvue with SHFR Generator



Have More Questions?

Browse our authorized dealer portal website. This site contains documentation to assist with installation, maintenance, and troubleshooting of Summit supported X-ray equipment.

SUMMITXRAY.COM>

LOCATE US

SUMMIT INDUSTRIES
7555 N CALDWELL AVE
NILES, ILLINOIS 60714

CONTACT US

tscalls@summitindustries.net
customerservice@summitindustries.net
[\(773\)588-2444](tel:(773)588-2444) or [\(773\)353-4000](tel:(773)353-4000)

