



SHFR Exposure Aborted Errors



MAIN TAKEAWAY

SHFR constantly monitors exposure conditions and aborts when safety or timing conditions fail.

Hey there Subscriber,

These following errors involve an exposure starting but not completing correctly.

- **Err 048 — Door Aborted Exposure**
Reset system; reboot Generator if needed
- **Err 050 — Aborted Exposure Error**
Reset system; reboot Generator if needed
- **Err 051 — Wrong Exposure Time Error**
Reset system; reboot Generator if needed
- **Err 071 — Interlock Error**
Reset system; reboot Generator if needed
- **Err 072 — Positioner OK Error**
Reset system; reboot Generator if needed
- **Err 073 — XON Feedback Error**
Reset system; reboot Generator if needed

■ Need full troubleshooting steps?

See the **SHFR Error Code Operator Guide** for detailed procedures:

Err 048 (p. 101) • Err 050 & 051 (p. 102) • Err 071 (p. 111) • Err 072 & 073 (p. 112)

Open SHFR Error Code Guide



Have More Questions?

Browse our authorized dealer portal website. This site contains documentation to assist with installation, maintenance, and troubleshooting of Summit supported X-ray equipment.

SUMMITXRAY.COM>

Congratulations to the
Field Service Engineers
who have completed
certifications:

Joshua Burgess on Nova
OTC/OTS & SHFR Generator

LOCATE US

SUMMIT INDUSTRIES
7555 N CALDWELL AVE
NILES, ILLINOIS 60714

CONTACT US

tscalls@summitindustries.net
customerservice@summitindustries.net
[\(773\)588-2444](tel:(773)588-2444) or [\(773\)353-4000](tel:(773)353-4000)

