




Tube Protection & Thermal Limits

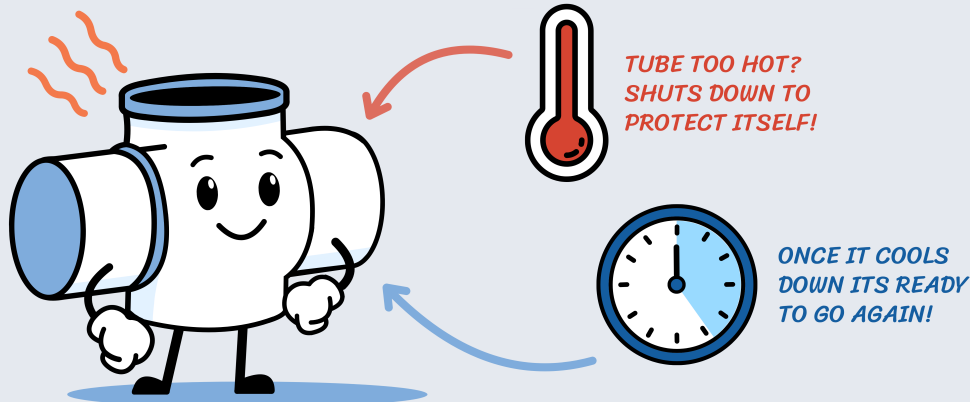

MAIN TAKEAWAY

Many "system down" calls are actually **thermal protection** events.

Hey there Subscriber,

The following errors involve overheating and cooldown-related protections.

- **Err 029 — Generator Overheat Error**

Allow Generator to cool; reduce exposure load if needed

- **Err 034 — Tank Presostat Error**

Turn Generator off and wait 30 minutes

- **Err 036 — Tube Thermostat Error**

Turn Generator off and wait 30 minutes

- **Err 037 — Tube Overload Error**

Tube protection active; wait for cooldown

- **Err 057 — Dual Energy Error**

Reset system; allow Tube/Generator cooling or lower technique

Need full troubleshooting steps?

See the **SHFR Error Code Operator Guide** for detailed procedures:

Err 029 (p. 86) • Err 034 (p. 89) • Err 036 (p. 90) • Err 037 (p. 91) • Err 057 (p. 105)

Open SHFR Error Code Guide



Have More Questions?

Browse our authorized dealer portal website. This site contains documentation to assist with installation, maintenance, and troubleshooting of Summit supported X-ray equipment.

SUMMITXRAY.COM>

Congratulations to the
Field Service Engineers
who have completed
certifications:

Mike Neaves on Nova
OTC/OTS & SHFR Generator

LOCATE US

SUMMIT INDUSTRIES
7555 N CALDWELL AVE
NILES, ILLINOIS 60714

CONTACT US

tscalls@summitindustries.net
customerservice@summitindustries.net
[\(773\)588-2444](tel:(773)588-2444) or [\(773\)353-4000](tel:(773)353-4000)

