


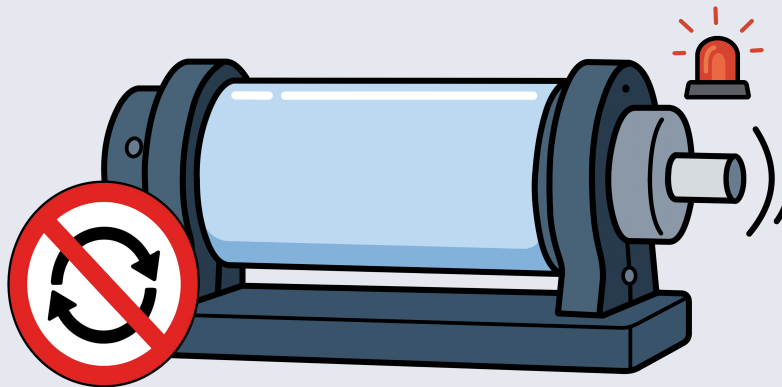

Rotor & Starter Errors on SHFR



MAIN TAKEAWAY

Rotor/ startup faults often point toward starter issues, tube issues, or speed-selection failures.

Hey there Subscriber,

These following errors involve rotor startup, tube readiness, overload conditions, or starter failures.

- **Err 018** — Rotor Error
{Reset; reboot; call service if needed}
- **Err 063** — Rotor Ready Error
{Reset; reboot; call service if needed}
- **Err 101** — No Speed Available
{Reset; reboot; call service if needed}
- **Err 102** — Low Speed Unavailable
{Reset; reboot; call service if needed}
- **Err 103** — High Speed Unavailable
{Reset; reboot; call service if needed}
- **Err 125–240** — Dual Speed Starter Failures
{Reset; reboot; call service if needed}

 **Need full troubleshooting steps?**

See the **SHFR Error Code Operator Guide** for detailed procedures:

Err 018 (p. 78) • **Err 063** (p. 108) • **Err 101** (pp. 116- 117) • **Err 102 & 103** (p. 118) • **Err 125- 240** (pp. 122- 162)

Open SHFR Error Code Guide



Have More Questions?

Browse our authorized dealer portal website. This site contains documentation to assist with installation, maintenance, and troubleshooting of Summit supported X-ray equipment.

SUMMITXRAY.COM>

Congratulations to the
Field Service Engineers
who have completed
certifications:

Jason Embrey on Nova
OTS/OTC & SHFR Generator

LOCATE US

SUMMIT INDUSTRIES
7555 N CALDWELL AVE
NILES, ILLINOIS 60714

CONTACT US

tscalls@summitindustries.net
customerservice@summitindustries.net
[\(773\)588-2444](tel:(773)588-2444) or [\(773\)353-4000](tel:(773)353-4000)

