




AEC & Dose Errors on SHFR

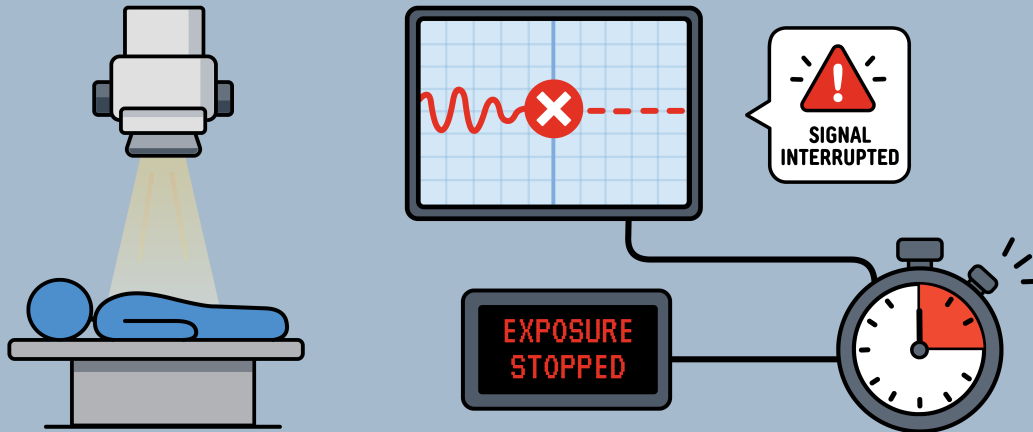

MAIN TAKEAWAY

AEC-related problems can appear as detector, calibration, or exposure timing failures.

Hey there Subscriber,

These following errors involve automatic exposure control and calibration-related issues.

- **Err 055 – Not Enough Dose Error**

Reset, check cables, reboot, call service if needed

- **Err 060 – Autocal Error**

Reset, reboot, call service if needed

- **Err 062 – AEC Error**

Reset, try different parameters, call service if needed

- **Err 070 – AEC Rapid Termination**

Reset, check cables, reboot, call service if needed

- **Err 104–109 – Dosimeter Errors**

Reset, check dosimeter/cables, reboot, call service if needed

Need full troubleshooting steps?

See the **SHFR Error Code Operator Guide** for detailed procedures:

Err 055 (p. 104) • Err 060 (p. 106) • Err 062 (pp. 107) • Err 070 (p. 111) • Err 104- 109 (pp. 119- 121)

SHFR Error Code List



Have More Questions?

Browse our authorized dealer portal website. This site contains documentation to assist with installation, maintenance, and troubleshooting of Summit supported X-ray equipment.

SUMMITXRAY.COM>

LOCATE US

SUMMIT INDUSTRIES
7555 N CALDWELL AVE
NILES, ILLINOIS 60714

CONTACT US

tscalls@summitindustries.net
customerservice@summitindustries.net
[\(773\)588-2444](tel:(773)588-2444) or [\(773\)353-4000](tel:(773)353-4000)

